


Kii® Dissecting Balloon Access Systems

SETUP GUIDE

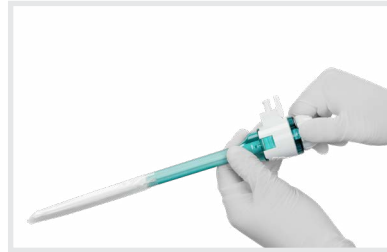


1 Insert the introducer until the introducer and seal are locked into place.



2 Insert the dissecting balloon into the incision site with the inflation port perpendicular to the abdomen.



3 Remove the introducer.

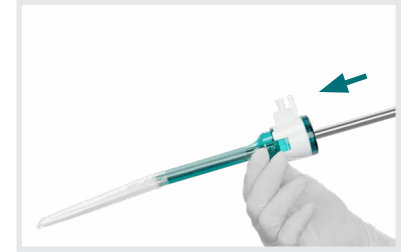


4 Insert a 10mm endoscope.



5 Connect the inflation bulb, and twist the Luer lock connector clockwise onto the stopcock until firmly attached.



6 Inflate the dissecting balloon under visualization by compressing the inflation bulb.

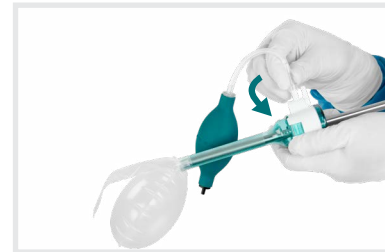

7 After completing dissection, untwist the Luer lock connector counterclockwise, and disconnect the inflation bulb.



8 Remove the endoscope for complete balloon deflation.

Visit [appliedmedical.com/dissectingballoons](https://www.appliedmedical.com/dissectingballoons) for more information.

This information is not intended to replace the Instructions for Use (IFU). Please refer to the IFU for the indications, contraindications, warnings, precautions, instructions and other information.